



INSTRUCTIONS: MITO ADAPT 3.0 FLOOR STAND

www.mitoredlight.com

Instructions: Mito**ADAPT 2.0** Floor Stand

Step 1: Unbox your stand and unpack the provided hardware.



Step 2: Thread the provided casters into the base of the floor stand and hand tighten (optional).



Step 3: Flip the stand onto its casters and install the 2 side braces and tighten using the provided screws with a Philips head screwdriver. (Ensure the holes line up as shown in the picture)



Instructions:

MitoADAPT 2.0 Floor Stand – Single Panel

Step 1: You will need to remove the lower two rubber feet from the back of your unit using a small Philips head screwdriver before installation into the floor stand.



Step 2: Insert your light into the Floor stand and ensure it is seated properly.



Important note: ensure the panel is fully seated into the bottom of the stand by pushing downwards on the top of the panel





www.mitoredlight.com



User Manual and Instructional Videos

This user manual and instructional videos can be found here: <https://mitoredlight.com/pages/user-manual>

Updated September 2023

Follow Us!



[/mitoredlight](https://www.facebook.com/mitoredlight)



[/mitoredlightofficial](https://www.instagram.com/mitoredlightofficial)



[@MitoRedLight](https://www.youtube.com/@MitoRedLight)



[@mitoredlightofficial](https://www.tiktok.com/@mitoredlightofficial)