



MITO PRO+ | MITO PRO X FLOOR STAND

Instructions: MitoPRO+ / MitoPRO X Floor Stand



Step 1: Unbox your stand and unpackage. You will find:


- 1 stand
- 4 wheel castors
- 2 panel locking screws



Step 2: Thread the provided casters into the base of the floor stand and hand tighten.

Wheels are optional, so skip this step if you prefer not to use them.

Instructions: MitoPRO+ / MitoPRO X Floor Stand

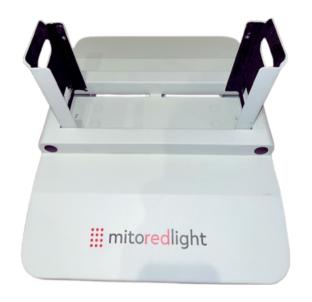

Step 3: Flip the stand onto its casters and flip up the two side braces.



Step 4: Add the two panel locking screws.

Instructions: Panel Placement



Step 1: You will need to remove the lower two rubber feet from the back of your unit using a small Philips head screwdriver before installation into the floor stand.



Step 2: Insert your light into the floor stand and ensure it is seated properly.

Important note: Ensure the panel is fully seated into the bottom of the stand by pushing downwards on the top of the panel.

Instructions: Panel Placement

Step 3: Use the two panel locking screws to secure the panel into place.



Step 4: Add power cord to panel and refer to the panel user manual on how to operate your panel.

Instructions: Wall mount

Your floor stand and panel can be secured to the wall for extra security. Your floor stand package will automatically come with the required equipment.




Step 1: Use the two panel locking screws to secure the panel into place.



Instructions: Wall mount

Step 2: Use the two panel locking screws to secure the panel into place.

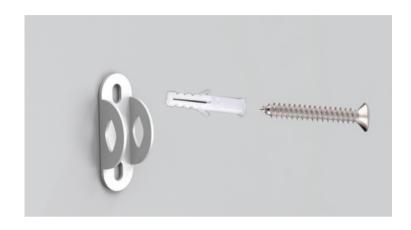

Step 3: Use the two panel locking screws to secure the panel into place.



Step 4: Use the two panel locking screws to secure the panel into place.





www.mitoredlight.com

User Manual and Instructional Videos

This user manual and instructional videos can be found here:

https://mitoredlight.com/pages/user-manual

Updated January 2025